# Lenire® Headphone Handbook

- DE Lenire®-Kopfhörer – Anleitung
- FR Manuel des écouteurs Lenire®
- IT Manuale d'uso delle cuffie Lenire®
- NL Lenire®-compatibele hoofdtelefoon

**LENIRE®**

# Lenire Compatible Headphones

The following document provides details on the use of the compatible Headphones supplied with the Lenire Tinnitus Treatment Device.

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The Lenire device is designed to work with specific Headphones. A list of compatible Headphones that can be provided at the time of purchase by your treating clinician is outlined below.

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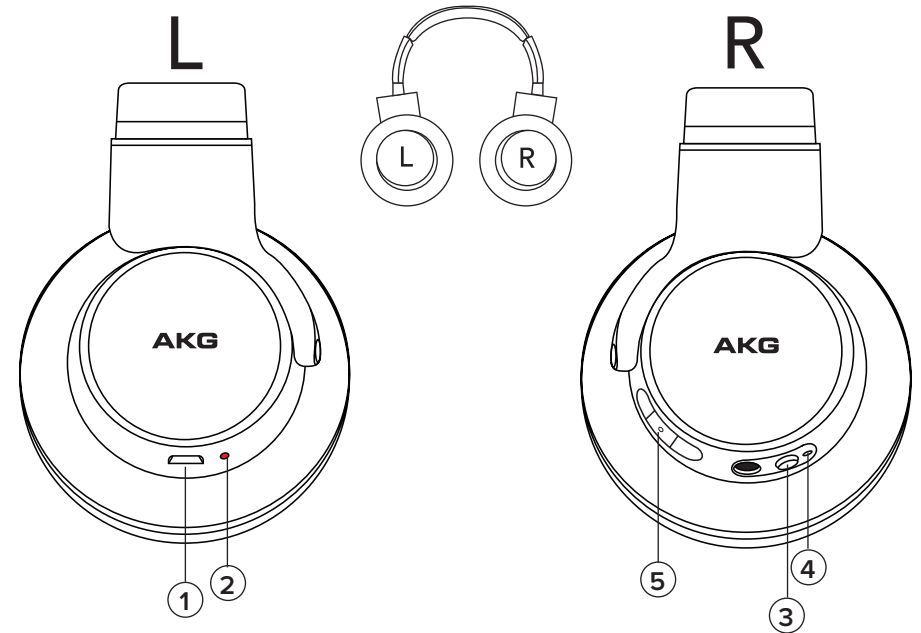
**5 Play/Pause & Volume Buttons (not used during treatment)**

- DE** - Wiedergabe/Pause- und Lautstärketasten (während der Behandlung nicht verwendet)
- FR** -Boutons Lecture/Pause et Volume (non utilisés pendant le traitement)
- IT** - Pulsanti di avvio/pausa e volume (non utilizzati durante il trattamento)
- NL** -Knoppen Afspelen/pauzeren en volume (niet gebruikt tijdens behandeling)

**1b : Connecting to the AKG K845BT / Verbinden mit dem AKG K845BT / Connexion à l'AKG K845BT / Connessione a AKG K845BT / Verbinding maken met de AKG K845BT**

Press the power button and wait for the Bluetooth Indicator to flash blue.

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Hold the Controller close to the Headphones.

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When connected the Headphones Indicator on the Controller will illuminate green.

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To disconnect bluetooth connection and enter Headphones into pairing-mode, hold the power button until the Indicator is rapidly flashing blue.

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The left and right sides of the Headphones are marked L and R respectively on the cloth inside the foam cushions.

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Plug the charging cable into the USB Port on the Headphones.

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When charging the Charging Indicator on the Headphones will illuminate red.

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Once charging is complete, the Charging Indicator on the Headphones will extinguish.

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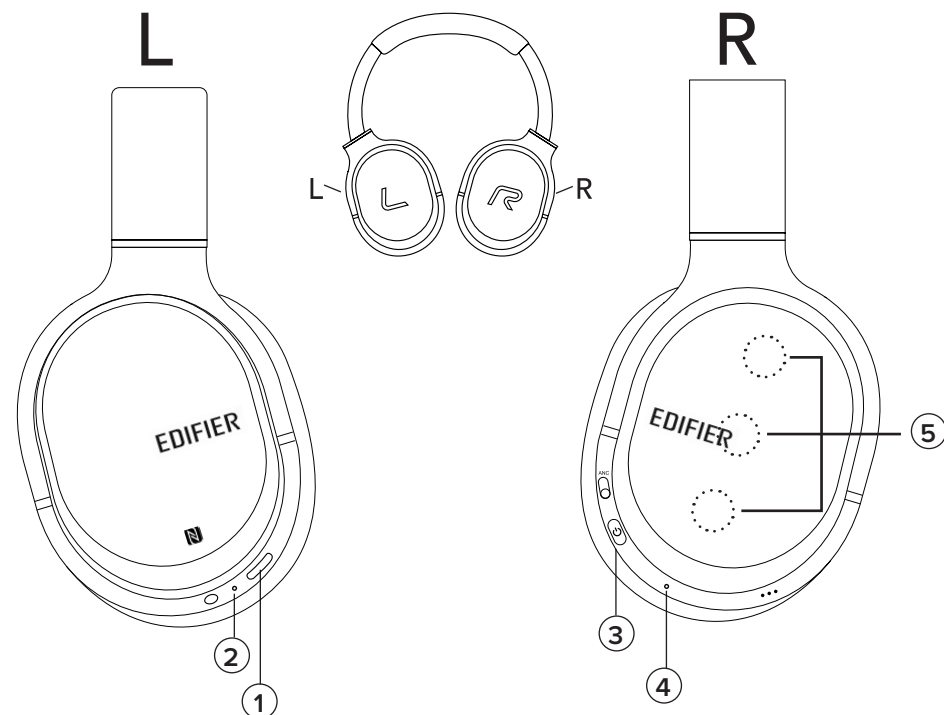
**5 Play/Pause & Volume Buttons (not used during treatment)**

- DE** - Wiedergabe/Pause- und Lautstärketasten (während der Behandlung nicht verwendet)
- FR** -Boutons Lecture/Pause et Volume (non utilisés pendant le traitement)
- IT** - Pulsanti di avvio/pausa e volume (non utilizzati durante il trattamento)
- NL** -Knoppen Afspelen/pauzeren in volume (niet gebruikt tijdens behandeling)

**2b : Connecting to the Edifier W860NB / Anschließén an den Edifier W860NB / Connexion à l'Edifier W860NB / Connessione a Edifier W860NB / Verbinding maken met de Edifier W860NB**

Press the power button for 3 seconds and wait for the LED Indicator to flash blue.

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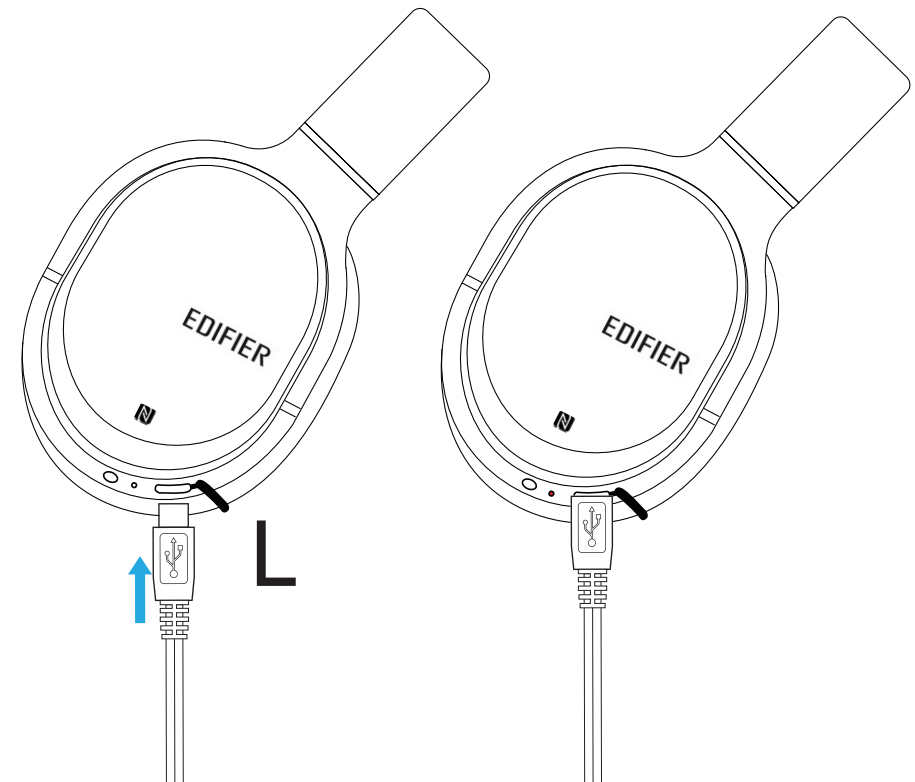
# Customer Support Contact Details

## General Contact Details

Neuromod Devices Limited,  
Rainsford St,  
Dublin,  
D08 R2YP, Ireland.

**E-mail:** [customersupport@neuromoddevices.com](mailto:customersupport@neuromoddevices.com)

**Web:** [www.neuromoddevices.com](http://www.neuromoddevices.com)

## Technical Support Contact Details

**E-mail:** [techsupport@neuromoddevices.com](mailto:techsupport@neuromoddevices.com)

**Web:** [www.neuromoddevices.com](http://www.neuromoddevices.com)

Neuromod Devices Ltd., Rainsford St., Dublin, D08 R2YP, Ireland.  
[www.neuromoddevices.com](http://www.neuromoddevices.com)

MM0111 V2.0