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
LENIRE®

Lenire® Tinnitus Treatment Device User Manual

- FR Dispositif de traitement des acouphènes Lenire®. Manuel de l'utilisateur
- DE Lenire®-Tinnitus-Behandlungsgerät. Benutzerhandbuch
- IT Dispositivo per il trattamento dell'acufene Lenire®. Manuale utente
- NL Lenire®-tinnitusbehandelingssysteem. Gebruikershandleiding

English *page 2*

Please read this manual carefully before using Lenire to ensure the device is used safely and effectively.

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Table of Contents

Using this Manual	2	Section D - Troubleshooting	13
Section A - General Information	3	Chapter 19 - Frequently Asked Questions	13
Chapter 1 - Introduction	3	Chapter 20 - Bluetooth Pairing / Un-Pairing	15
Chapter 2 - Intended Use	3	Chapter 21 - Servicing	15
Chapter 3 - Contraindications	3	Chapter 22 - Device Errors	15
Chapter 4 - Risks Associated with Use	3	Section E - Technical Information	16
Chapter 5 - General Warnings	3	Chapter 23 - Environmental Operating Conditions	16
Chapter 6 - Product and Package Labelling	4	Chapter 24 - Radio Frequency (RF) Specifications	16
Section B - Using Lenire	5	Chapter 25 - Guidance and Declaration for Electromagnetic Compatibility	16
Chapter 7 - Product Components and Accessories	5	Chapter 26 - Materials in Direct Contact with Human Tissue	17
Chapter 8 - Controls and Indicators	5	Chapter 27 - Declaration of Conformity Radio Equipment Directive 2014/53/EU	17
Chapter 9 - Controller Port	7	Chapter 28 - Headphone Specific Technical Information	17
Chapter 10 - Charging the Controller and Headphones	7	Section F - Customer Support Contact Details	18
Chapter 11 - Device Setup	8		
Chapter 12 - Tonguetip Setup	9		
Chapter 13 - Starting and Controlling a Treatment Session	10		
Chapter 14 - Treatment Guidelines	11		
Section C - Care, Storage, Maintenance and Disposal	11		
Chapter 15 - Cleaning	11		
Chapter 16 - Storage and Transport	12		
Chapter 17 - Maintenance	12		
Chapter 18 - Disposal	13		

Using this manual

This manual is supplied with the Lenire device to provide you with information on the product, treatment regime and instructions for its safe and effective use. You should read and understand these instructions before use.

Section A

General Information

This section outlines the general information relating to the Lenire device, including important safety information.

Chapter 1 - Introduction

Lenire® is a medical device intended to reduce the symptoms of tinnitus. It comprises a handheld Controller and a Tonguetip® which, delivers gentle electrical stimulation to the tongue and is used in conjunction with wireless Headphones that deliver sound stimulation. A Charger is included that can be used to charge both the Controller and the Headphones.

The sound and tongue stimulation are configured and calibrated to your individual characteristics during the initial fitting procedure by a trained healthcare professional.

Chapter 2 - Intended Use

The Lenire device is intended to be used by tinnitus sufferers of at least 18 years of age to alleviate the symptoms of chronic, subjective tinnitus. It should be used for 30 to 60 minutes per day, every day, for at least 10 weeks, after which you may experience sustained alleviation of your tinnitus symptoms.

The Lenire device can be used in the home environment after consultation and fitting by a suitably qualified healthcare professional.

Lenire is intended for prescription use only. Use by a person for whom the device has not been configured may cause discomfort, or temporary exacerbation of tinnitus loudness.

Chapter 3 - Contraindications

 **Caution!** – Do not use the device if:

- You have a pacemaker, defibrillator or any other active implantable device, unless directed by a doctor.
- You are pregnant, unless directed by a doctor.
- You suffer from epilepsy or any other condition that may result in loss of consciousness.
- You suffer from any condition that causes impaired sensitivity of the tongue.
- You have lesions, sores, or inflammation of the oral cavity, unless directed by a doctor.
- You are suffering from any intermittent or chronic neuralgia in the head and neck area.

Chapter 4 - Risks Associated with Use

Potential risks associated with use of the Lenire device include:

-  Contact between the Tonguetip and dental retainers or metal fillings may cause temporary discomfort.
-  There is a possibility that some people may experience a mild allergic reaction to the materials in the device.
-  You may experience a mild numbing sensation or a discomfort on the tongue during treatment. If this occurs, reduce the tongue stimulation. If the symptoms persist, contact your prescribing healthcare professional.
-  Use of the treatment may result in mild fluctuations in tinnitus symptoms, such as tinnitus loudness.
-  Headaches may accompany use of the treatment device.

Chapter 5 - General Warnings

-  Do not use the Tonguetip when going to sleep or under the influence of alcohol or other substances that cause drowsiness.
-  Keep the Tonguetip and Charger out of the reach of children, to avoid possible choking and strangulation hazards.
-  To avoid the risk of cross-contamination, the Tonguetip should not be shared with any other person, as it is a single-patient, multi-use device.
-  Do not use the Tonguetip if you have lesions, sores, or inflammation of the oral cavity. Discontinue use of the device until symptoms have resolved.
-  Do not leave the device in the sun, near or in an open fire or on a radiator.
-  The use of rechargeable Lithium Ion batteries is very common and they are generally considered safe to use. With any energy storage device, they carry safety risks including overheating. Only use the Charger provided with your device and always follow the instructions in this manual when charging your Lenire device or Headphones.
-  Do not attempt to replace the lithium-ion batteries in your Controller or Headphones. Attempts to replace the batteries may result in risks including overheating and fire.

 To avoid the risk of a burn do not puncture, crack or crush the Controller or Headphones.

 Do not use in conjunction with a hearing aid.

Chapter 6 - Product and Package Labelling

The label on the Controller, the Tonguetip plug, Carry Case and the product packaging provide important information. The meanings of the associated symbols are described below, and in the context of the relevant sections throughout this User Manual.

 Medical Device.




 Use-by date (year, month and day, YYYYMMDD). This is the date after which the device should no longer be used.

 Do not use if the packaging is damaged or previously opened.

 Consult the User Manual.

 This symbol indicates the need for the user to consult the User Manual for warnings and precautions associated with this medical device.

 Manufacturer's details (name and address) and date of manufacture (year, month and day, YYYYMMDD).

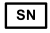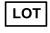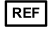 The device's Serial Number, Lot Number and Model Number appears next to these symbols.
 You will need these numbers when requesting assistance from your prescribing healthcare professional (refer to Section F).

Ingress Protection (IP) Rating: IP ratings describe how effectively an enclosure is sealed.

IP22 IP22 rated equipment is protected from objects greater than 12 mm in diameter being inserted e.g. fingers, and against small amounts of vertically dripping water.

 Non-ionizing radiation.

 This symbol signifies that the device should not be disposed of as domestic waste. This ensures compliance with the Battery Directive 2006/66/EC.

LENIRE The logo for this product range (Lenire).

 This symbol, the CE Mark, signifies compliance with applicable EU Directives.

The number (2797) is the Notified Body number for the British Standards Institute (BSI), who approved Lenire for CE Marking.

 Manufacturing symbol to aid identification of the device SKU.

Rx only Caution: Federal law restricts this device to sale by or on the order of a qualified audiologist or physician. (US Only).

Section B

Using Lenire

This section describes the components, controls and indicators of Lenire and outlines the steps required to prepare for and commence treatment.


Chapter 7 - Product Components & Accessories

In the Lenire device box, you will find the three main components of your Lenire device:

Controller

The Controller (Model Number CR-201) is used to “control” the treatment session i.e. you can start, pause and resume the treatment session, adjust the volume of the sound, adjust the level of the tongue stimulation, and turn the device on or off.

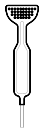Tonguetip

The Tonguetip (Model Number TP-101) is a device which provides gentle, electrical stimulation to the tongue.

Headphones

The Headphones (Model Number Hi-X25BT) deliver sound stimulation to the ears.

Also included in the box:

Charger

The supplied Charger is used to charge the Controller and the Headphones.

Carry Case and Bag


The Carry Case and Accessory Bag can be used to store your Lenire device inbetween uses.

Quick Start Guide

The Quick Start Guide outlines the basic steps needed to commence treatment.

7.1 - Headphones

Lenire must be used in conjunction with a set of Bluetooth Headphones. For best performance, it is essential that you use the supplied headphones. See chapter 29 for manufacturer details and technical information.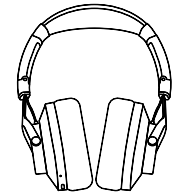

Note: Lenire is designed to work only with the supplied Headphones, and will not function with any other Bluetooth devices.

For information and specific instructions on other supported headphones, please visit www.lenire.com/product-support/ to download the Lenire Headphone Handbook.

7.2 - Batteries

The Controller and Headphones are powered by 3.7V Lithium Ion batteries. The batteries are pre-fitted and are not intended to be replaced. See Chapter 10 for instructions on charging the Controller and Headphones.

7.3 - Charger

The supplied Charger should be used to charge the Controller and may also be used to charge the Headphones in conjunction with the adaptor provided.

The supplied compatible Chargers are:

Manufacturer: XP Power


Model No.: ACM06US05W-XZ1663A or VEP08US05-XE0767

If experiencing difficulties with charging the Controller or the Headphones, consult Chapter 19 Troubleshooting or refer to Section F.

Chapter 8 - Controls and Indicators

8.1 - Treatment Controls

There are three control buttons on the front of the Controller: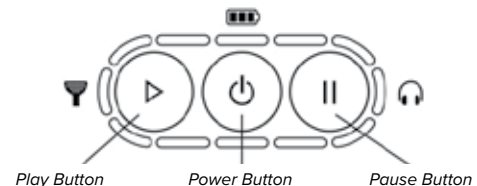

1. Power Button

Hold this button for 2 seconds to switch the device on.
Hold this button for 2 seconds to switch the device off.

2. Play Button


Press this button to start or resume the treatment session.

3. Pause Button

Press this button to pause the treatment session.

8.2 - Indicators

The status of the device is indicated by several lights surrounding the buttons on the top of the Controller.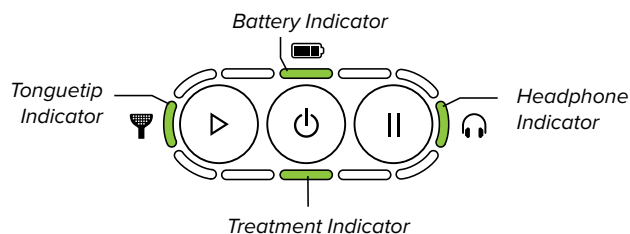

Battery Indicator 	
Green	The battery power is ok.
Orange	The battery has less than 1 hour of power remaining. Please charge the device before next use (see Chapter 10).
Red	The battery is exhausted. The device will shut off automatically. Please charge the device before next use (see Chapter 10).
Orange (Pulsing)	The Charger is connected and the battery is charging.
Green (Pulsing)	The Charger is connected and the battery is fully charged.

Headphones Indicator 	
Blue (Pulsing)	The Controller is trying to establish a wireless connection to the Headphones.
Green	The Controller is connected wirelessly to the Headphones.
Blue (Pulsing Rapidly)	The Controller is attempting to pair with the Headphones, see Chapter 20 for details.
Orange (Pulsing Rapidly)	An unsupported headphone is connected to the Controller.

Tonguetip Indicator 	
Blue (Pulsing)	There is no Tonguetip connected.
Green (Pulsing)	The Tonguetip is connected but it is not in contact with the tongue.
Green	The Tonguetip is connected and it is in contact with the tongue.
Orange or Orange (Pulsing)	The Tonguetip has less than 10 hours of use remaining after which it will no longer function. A new Tonguetip is required to avoid disruption to your treatment regime.
Red	The Tonguetip has expired and will no longer function. Please purchase a new Tonguetip to continue treatment.

Treatment Indicator	
White (Pulsing)	The treatment session is paused.
White (Animated)	The treatment session is active – the white light rotates around the buttons.
Green (Pulsing)	The treatment session has completed.


Note: When the device is powered off, none of the lights will be illuminated.

8.3 - Headphones Volume and Tonguetip Stimulus Controls


Tonguetip Stimulation Control

The Tonguetip stimulation intensity can be adjusted to a comfortable level by pressing the buttons indicated below. Please see Chapter 13.3 for guidelines on setting the stimulation intensity.

Left Side View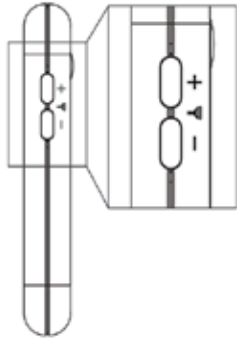

Indication of default (optimum) Tonguetip stimulus intensity

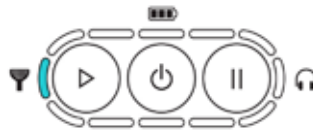Indication of maximum Tonguetip stimulus intensity


Indication of minimum Tonguetip stimulus intensity


Headphones Volume Control

The Headphones volume can be adjusted to a comfortable level by pressing the buttons indicated below. Please see Chapter 13.3 for guidelines on setting the volume level.


Right Side View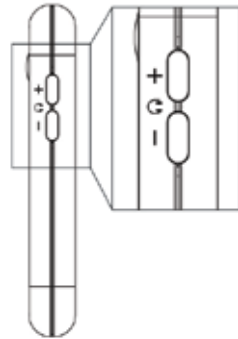

Indication of default (optimum) Headphones volume level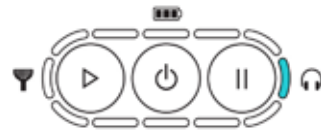

Indication of maximum Headphones volume level

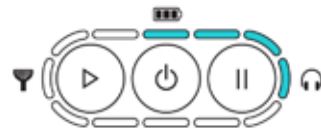Indication of minimum Headphones volume level


Note: The Headphones volume and Tonguetip stimulation are calibrated to the optimum levels during device fitting by your healthcare professional. For further details, please see Chapter 13.3.

Chapter 9 - Controller port

The Controller Port is used to connect the Tonguetip and the Charger to the Controller. For safety, the Tonguetip and the Charger cannot be connected to the Controller at the same time.

The following symbols are adjacent to the Controller Port:

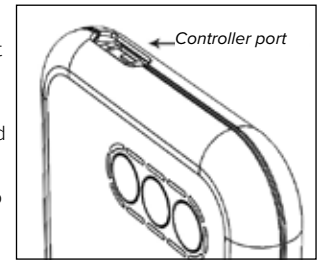This symbol (direct current) identifies a charging port.

This symbol represents the Tonguetip.

The USB symbol – this is the connection that is used by the healthcare professional when they configure your device.

This symbol indicates that the Tonguetip provides a high degree of protection against electric shock.

The Tonguetip and the supplied Charger are the only equipment that should ever be plugged into the Controller Port.

Chapter 10 - Charging the Controller and Headphones

Before Lenire is used for the first time, it is recommended that the Controller and the Headphones be fully charged.

Only use the supplied Charger with the Lenire device.


To avoid the risk of electric shock, do not connect or disconnect the Charger from the mains if it, or your hands, are wet.

10.1 - Charging the Controller


To charge the Controller:

- Plug the cable end of the supplied Charger into the Controller Port of the Controller.
- Plug the Charger into a mains electrical socket.


While charging, the Battery Indicator will pulse orange. Once the battery is fully charged, the Battery Indicator will pulse green.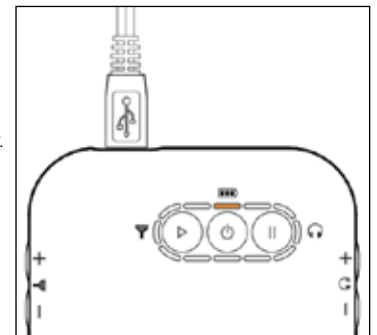

Battery charging

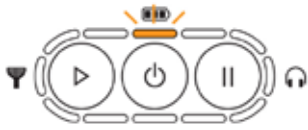Battery charged

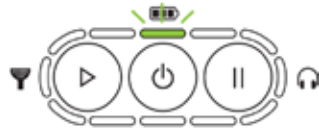
Note: An empty battery should take no more than 3 hours to fully recharge, and a full charge should allow the Lenire device to be used for at least 8 hours.

Note: Once the battery is fully charged, the Charger should be disconnected to help conserve the life of the battery in the Controller.

10.2 - Charging the Headphones

To charge the Headphones:

- Plug the cable end of the supplied Charger into the USB-C® adaptor provided, then insert into the USB-C® Port of the Headphones.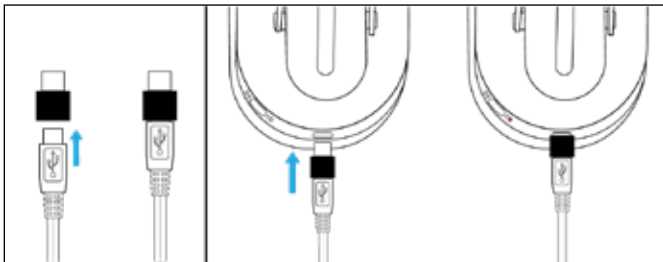


- Plug the Charger into a mains electrical socket.


Alternatively you can charge the Headphones using the separate USB-C® cable provided in the packaging. For convenience a USB-A adaptor has also been provided to increase compatibility options for this cable.

While charging, the light on the Headphones will illuminate red. Once the battery is fully charged, the light on the Headphones will turn off.

Battery charging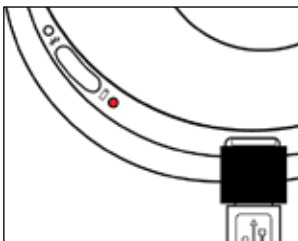

Battery charged

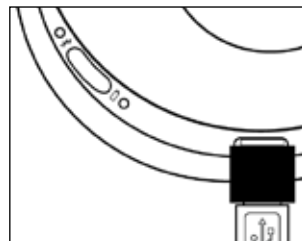Note: When the Headphones battery is running low, the battery LED will blink red and the Bluetooth LED will blink blue, you will also hear an audio prompt.

Note: An empty battery should take no more than 3 hours to fully recharge, and a full charge should allow the Headphones to be used for at least 10 hours.

Note: Once the battery is fully charged, the Charger should be disconnected to help conserve the life of the battery in the Headphones.

For details on your specific Headphones, please see the Lenire Headphone handbook at www.lenire.com/product-support/.

Chapter 11 - Device Setup

To commence a treatment session, the Lenire device should be set up as follows:

11.1 - Controller and Headphones Setup

The Headphones should be worn so they cover both ears and should be adjusted to maximise comfort before commencing the treatment session.

Note: To adjust the headphone's size, please hold the headband on the glossy parts and slide the headbow carefully outwards/inwards. Please handle with care.

! It is essential that the left and right earphones cover the left and right ears, respectively. This is because

the device calibration is unique to each ear – it is based on your audiogram. Failure to put on the headphones correctly could result in excessive loudness in one ear, which could cause discomfort, or incorrect loudness that may reduce the effectiveness of the treatment.


The left and right sides of the headphones are marked L and R respectively on the cloth inside the foam cushions.

11.2 - Connecting the Headphones with the Controller

The Controller connects to the Headphones wirelessly via Bluetooth by following these steps:

- Turn on the Controller by pressing its Power Button for two seconds. The Controller will attempt to connect to the Headphones during which time the Controller's Headphones Indicator will pulse blue.
- Press the Headphones' power button for 2 seconds and wait for the LED Indicator to pulse blue (or blue and red if the Headphones battery is low). See the Lenire Headphone Handbook at www.lenire.com/product-support/ for Headphone specific details.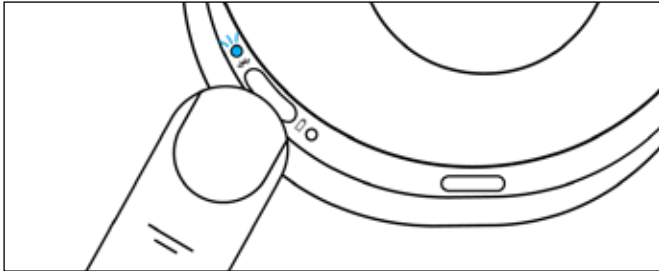

- Once the connection to the Headphones has been established, the Controller's Headphones Indicator will illuminate green.

Note: The above instruction assumes that the Headphones are already 'paired' with the Controller. This should be the case in most situations, as your healthcare professional will have paired the Headphones during the device fitting. If they are not paired, or if you are having difficulty connecting to the Headphones, then you should follow the instructions in Chapter 20.

Note: During treatment, the Controller can be kept adjacent to you, e.g. in a pocket or a bag, or in another dry location that is close by, e.g. on a bedside table.


Chapter 12 - Tonguetip Setup

The Tonguetip is supplied within a plastic Carry Case for storage and transportation purposes.

! If the Tonguetip is damaged, it should not be used and should be returned to the point of sale.

Note: Before connecting and using the Tonguetip for the first time, it should be cleaned as per the instructions in Section C.

The Tonguetip must be connected to the Controller by plugging it into the Controller Port, such that the label side of the plug faces away when viewed from the front of the Controller.

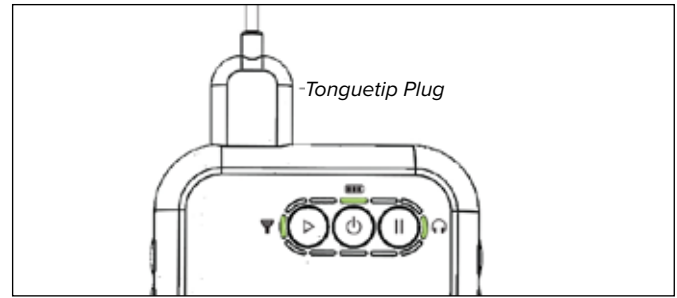Once connected, the Tonguetip Indicator should change from pulsing blue to pulsing green.

The Tonguetip should be placed in your mouth, just behind your upper front teeth (on the top of the tongue towards the front of the mouth) with the electrodes facing down.

As you close your mouth, the electrodes should make contact with the tip of your tongue and rest there comfortably.

Once adequate contact is made with the tongue, the Tonguetip Indicator should stop pulsing (it should illuminate green without pulsing).

For maximum comfort, the mouth and jaw should remain closed but relaxed for the duration of the treatment session.


! To avoid the risk of electrical shock or burns, do not connect the Controller or Tonguetip to any equipment other than those specified in this User Manual.

Note: The Tonguetip model TP-101 is designed to function for 360 treatment sessions (180 hours) before expiring. Indications of imminent expiry and actual expiry are outlined in Chapter 8.

Chapter 13 - Starting and Controlling a Treatment Session

13.1 - Starting a Treatment Session

As soon as the Headphones and the Tonguetip are connected, which will be indicated by the Headphones Indicator and the Tonguetip Indicator illuminating green, you are ready to start a treatment session.

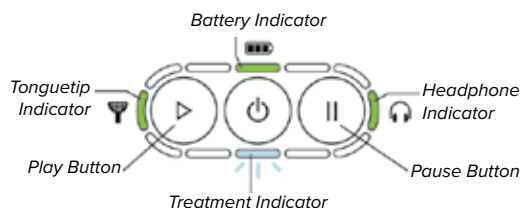Once you are in a comfortable position, press the Play Button to commence the treatment. The Treatment Indicator (white) will rotate around the display.

A single treatment session lasts approximately 30 minutes from the time the Play Button is pressed.

Towards the end of the treatment session, the sound will gradually fade out over a period of 2 minutes to a lower volume level.

! It is essential that you continue to listen to the sound during the fade out period, i.e., until the treatment session is complete.

Once the treatment session is complete, the Treatment Indicator will pulse green.

Note: The Controller will power off automatically after 20 minutes of being idle (no treatment session in progress). The Headphones will automatically power off 5 minutes thereafter. It is recommended to power off the Controller and the Headphones after the treatment session has completed.

13.2 - Pausing and Resuming a Treatment Session

It is recommended to complete each treatment in a single, continuous 30-minute session to maximise the effectiveness of the treatment.

However, the treatment session can be paused at any time by pressing the Pause Button. The Treatment Indicator will pulse white instead of rotating around the display.

The treatment session will automatically pause if the Tonguetip loses contact with the tongue for more than 30 seconds.

Note: Pausing the treatment session does not affect the total duration of active stimulation. When treatment is resumed it will provide the remainder of the 30 minutes of stimulation.

To resume treatment, press the Play Button while also ensuring that the Tonguetip is in contact with the tongue.

If you do pause a treatment session, it is recommended that you resume again within 5 minutes. It is also recommended that you do not pause the treatment more than once within a single treatment session in order to maximise the effectiveness of the treatment.

13.3 - Adjusting the Stimulus Levels

13.3.1 - Tonguetip Stimulus Level Control

It is recommended to leave the tongue stimulus level set to the default level unless either:

- There is a strong sensation from the Tonguetip that causes discomfort. In this case the tongue stimulus level can be adjusted downwards to a comfortable level.
- There is a weaker tongue sensation than was perceived during the initial training you received as part of the device fitting. In this case the tongue stimulus level may be adjusted upwards.

Note: The tongue stimulation is calibrated to your sensation thresholds during device fitting by your healthcare professional. The treatment stimulation is then set to be sufficient for optimum effectiveness and may not be perceptible. If the stimulation level is set to the default level (i.e. the level that the device defaults to at the start of each treatment session), then the stimulation is functioning properly as long as the Tonguetip Indicator is illuminated green (i.e., when it is not pulsing).

13.3.2 - Headphones Volume Control

The Headphones volume level should be left at the default level unless either:

- The treatment environment is noisy, in which case the volume level can be increased.
- The volume is so high that it causes any discomfort, or prevents you from relaxing, in which case the volume level can be decreased. If the sound causes discomfort even when set to the lowest volume level, please pause treatment and contact your healthcare professional.

Note: The Headphones volume levels are calibrated to your hearing thresholds for optimum effectiveness.

Note: Depending on your audiological profile, you may perceive the sound levels to be soft. It is important that you seek a quiet environment for your treatment sessions.

Note: Depending on your hearing profile, the range over which you can increase the Headphones volume level may be limited. This is to ensure that the volume levels remain within safe limits so as not to cause hearing damage.

Chapter 14 - Treatment Guidelines

The following guidelines should be followed to maximise the effectiveness of the treatment programme:

- While the treatment is intended for use 30 to 60 minutes per day, it is recommended that you try to complete at least a full 30 minutes at each daily treatment.
- Try to keep the tongue stimulation level and the Headphones volume level at the default settings, as these are the optimum levels calibrated to your individual characteristics.
- Try to complete each treatment session in a place that is quiet, relaxing and comfortable. Focusing your attention away from distractions such as the TV may improve the effectiveness of the treatment.
- Try to keep a routine for the treatment sessions, where the treatment is delivered at the same time(s) each day.
- It is not unusual for tinnitus symptoms to disimprove or fluctuate. It is recommended that you continue treatment even during periods where the symptoms disimprove, for the 10 weeks minimum recommended treatment duration. However, if there is a significant disimprovement in your symptoms it is recommended to seek advice from your prescribing healthcare professional.

 Do not use whilst showering, in rain, or in a moist atmosphere such as a steam bath or sauna. Do not use whilst applying items such as perfume, cosmetics, after-shave, hairspray or suntan lotion.

Section C Care, Storage, Maintenance and Disposal

This section details the correct procedures for cleaning, storage, maintenance and disposal of Lenire.

Chapter 15 - Cleaning

It is advisable to clean the Controller, the Headphones, the Carry Case and the Tonguetip after prolonged disuse or storage, or after exposure to conditions that could affect their hygiene.

15.1 - Instructions for Cleaning of the Controller, the Headphones and the Carry Case

 Before cleaning the Controller or Headphones, ensure that they are switched off and not connected to the Charger.

- The exterior surfaces of the Controller, the Headphones and Carry Case should be cleaned using a soft cloth to remove debris.

15.2 - Instructions for Cleaning of the Tonguetip

It is recommended to follow the procedure below to clean the Tonguetip prior to first use, and prior to every subsequent re-use.

 Before cleaning the Tonguetip ensure that it is disconnected from the Controller.

 Do not expose the Plug end of the Tonguetip to liquids.

- The Tonguetip should be rinsed thoroughly with water or distilled water.
- You may use the Tonguetip immediately after rinsing.
- If you do not intend to use the Tonguetip immediately after cleaning, place on a clean, dry surface and allow it to air-dry.
- In addition, the Tonguetip may be cleaned by gently rubbing the Tonguetip surfaces, including the electrodes and the plug using 70% IPA (Isopropyl alcohol) wipes.

Note: 70% IPA wipes are available over the counter in pharmacies.


- ⚠ After cleaning the tonguetip with IPA, place on a clean, dry surface to allow to air-dry for at least one minute before use.
- ⚠ Do not use any cleaning agent other than IPA or water.
- ⚠ Attempting to clean the device using a method other than that described here can damage the device and voids the warranty.
- ⚠ Do not scrub or wipe the surfaces forcefully.
- ⚠ Do not immerse the Controller, the Charger or the Tonguetip plug in water.
- ⚠ To avoid damaging the devices do not force dry, for example by using a hairdryer, or leaving on top of a heater.

Chapter 16 - Storage and Transport

16.1 - Carry Case and Accessory Bag

It is recommended to store the Tonguetip in the Carry Case supplied as it has been specially designed to protect it during storage and transport. It can also help maintain the cleanliness of the Tonguetip in-between uses.

The Carry Case is designed to hold the Tonguetip and its cable within the body of the case. It is not necessary to disconnect the Tonguetip plug from the Controller port before placing it into the Carry Case. For ease of storage, it is possible to leave the Tonguetip cable outside the case, whereby the cable enters and exits the case through the holes as indicated in this image:

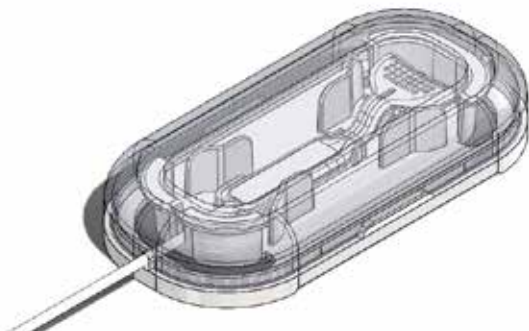An Accessory Bag is provided in the Lenire packaging. The bag will allow you to store the Controller, Tonguetip and Headphones between uses and aid in the transport of your device.

To fit the Headphones in the bag you can either turn the earcups backwards, so they lay flat, or you can fold the earcups towards the headband for very compact dimensions. Please handle with care.

16.2 - Storage and Transport Information

Note: The Controller has a shelf life of two years, meaning that it should be used for the first time within two years of it being manufactured, otherwise the battery life may deteriorate.

The environmental limits and precautions pertaining to storage and transport of the device are illustrated on the packaging labels, next to these symbols:

 Storage and transport temperature limits. The lower limit is -10°C, as indicated near the lower horizontal line and the upper limit, 45°C, as indicated near the upper line.

 Storage and transport humidity limits. The lower limit, 5%, as indicated near the lower horizontal line and the upper limit, 85%, as indicated near the upper line.

 Keep dry.

 Fragile, handle with care.

Chapter 17 - Maintenance

17.1 - Inspections

- Carry out a weekly visual inspection to ensure that the Controller has not been damaged.
- Carry out a weekly visual inspection to ensure there is no evidence of discolouration or damage to the Tonguetip.

Note: If you have any concerns following these inspections, cease using the device and request assistance from your prescribing healthcare professional (refer to Section F).

17.2 - Controller Service Life

- The Controller has an expected service life of three years. Beyond three years, the device will still perform as intended except that the battery capacity may start to deteriorate.
- The Controller should be charged every 6 months if not in use. Not charging the battery for extended periods may reduce the capacity of the battery and shorten the service life of the device.

17.3 - Tonguetip Service Life

- The Tonguetip has an expected service life of three years. It is recommended not to use the Tonguetip after this time period as the electrodes may not function as effectively.

- The Tonguetip will expire after 180 cumulative hours of actual use. If an expired Tonguetip is connected to the Controller the Tonguetip Indicator will flash red and treatment sessions will not commence until a non-expired Tonguetip is connected. A new Tonguetip (Model TP-101) can be purchased from your prescribing healthcare professional (refer to Section F).

Chapter 18 - Disposal

Please consider your personal safety and the environment when disposing of Lenire. When the device, or any of its components have reached the end of their service life, they should be disposed of in accordance with local regulations or returned to the manufacturer (see Section F).

No part of Lenire should be disposed of as domestic waste.

Section D Troubleshooting

The following section describes problems commonly encountered while using the Lenire device, and their most common solutions. If a particular problem cannot be solved using the information provided in this section, please contact your prescribing healthcare professional.

Chapter 19 - Frequently Asked Questions

19.1 - What should I do if the Controller will not power on?

- Ensure that the Controller is charged (see Chapter 10).
- Ensure the Charger is not connected. The device will not power on while the Charger is connected.
- Make sure you hold the Power Button for at least 2 seconds when attempting to turn on the device.
- It is possible that the Controller is in a state that requires a reset – connect the Charger briefly to reset the Controller and then try to turn the Controller on again once the Charger has been disconnected.

19.2 - What should I do if there is no sound coming from the Headphones?

- Ensure the Headphones are charged (see Chapter 10).
- Ensure the Headphones are paired to the Controller. If they are not, pair the Headphones with the Controller again (see Chapter 20).
- Check that the Headphones connect to the Controller – the Headphone Indicator should illuminate green when they are connected.
- It is important to confirm that the Headphones are what the Controller is connected to (i.e. it is possible that your Controller is inadvertently connecting to another device). You can verify this by turning the Headphones off while the Controller is still on, and the Headphones Indicator on the Controller should revert to pulsing blue.
- Please ensure that the Tonguetip is connected and that it has not reached expiry i.e. the Tonguetip Indicator is illuminated either green or orange, but not red.
- Ensure that the treatment session is active by pressing the Play Button. The Treatment Indicator will rotate around the light display when the treatment session is active.

19.3 - What should I do if the sound level is too low?

- Try adjusting the volume level using the buttons on the side of the Controller and using the device in a quiet location. See Chapter 13.3 for guidelines on adjusting the volume level.

Note: It is likely that the sound levels will be soft, depending on the device configuration and your audiological profile.

Note: The sound volume is calibrated to the optimum levels appropriate to your hearing thresholds during the device fitting with a trained healthcare professional. The range of adjustment above and below this optimum level is limited in order to maximise the treatment effectiveness and minimise possible hearing damage.

19.4 - What if I find the tongue stimulation level is too low?

Note: It is possible that there may be little or no perceived sensation from the Tonguetip (see Chapter 13.3).

- Ensure that the Tonguetip Indicator illuminates green (non-pulsing) when the Tonguetip is making contact with your tongue.
- Ensure that the Tonguetip status indicator is not flashing red.
- Try adjusting the stimulation level using the buttons on the side of the Controller.


19.5 - What should I do if the Controller is not charging?

If your Controller is not charging, please try the following:

- Check the mains plug on the Charger is fully 'clicked in'.

- Check if the Charger is working by connecting the Charger to the Headphones. If the Headphones do not indicate they are charging, then there is possibly a fault with the Charger.
- Retry charging the Controller.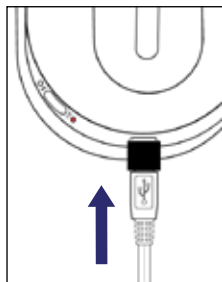

19.6 - I tend to fall asleep during the treatment - Is this a problem?

Falling asleep during a treatment session is not advised. It is important that the Tonguetip remains in place and contacting your tongue throughout the treatment session.

19.7 - I only received one Charger in the box, is that correct?

Yes. The same charger may be used to charge the Controller and the Headphones. See Chapter 10 for details.

19.8 - What should I do if I miss a treatment session?

To maximise the effectiveness of the treatment, it is important to adhere to the treatment regimen prescribed by your healthcare professional. If you miss a treatment session on any particular day, it is advisable that you continue treatment as normal the following day.

19.9 - My Headphones won't connect, even though the light on the Headphones indicates that it is connected

It is possible that another device, such as a mobile phone or a personal computer, is connecting to your Headphones thereby preventing the Controller from connecting to them. Try either:

- Turning off any mobile phones, or other computer devices that are close by.
- Moving to a different location, away from any computer devices.

19.10 - My Headphones touch controls have re-activated

Touch controls have been deactivated on the headphones by default for use with Lenire. In cases of accidental re-activation of the touch controls on the Headphones, please follow the steps below.

To confirm touch control activation:

- Power the Headphones on by pressing the power button until the "power on" voice command is audible.
- Power on the Controller and confirm connection with a solid green light next to the Headphone icon.
- Begin a treatment session by pressing play and tap the touch control on the right earphone. If a beep is audible, whenever the touch control is tapped, the touch control is activated.

To de-activate Touch Controls:

- Power off the Headphones by pressing and holding down the power button (you will hear the "power off" voice command), do not release the button.
- Continue to hold down the power button for 15 seconds then release the power button.
- Next power the Headphones back on, begin a treatment session by pressing play on the Controller, and tap the touch control on the right earphone, if no beep is audible, the touch controls are now deactivated.


Chapter 20 - Bluetooth Pairing / Un-Pairing

The Controller sends sound to the Headphones wirelessly using Bluetooth, and for this to work, the Controller and the Headphones need to be 'paired', allowing the two devices to connect automatically henceforth.


20.1 - Pairing Headphones to the Controller

The first time you try to use the Controller and Headphones together you may need to pair the devices as follows:

- Put the Headphones into Pairing Mode by first ensuring they are powered off, then hold the power button until the indicator adjacent to the power button rapidly flashes blue (see Lenire headphone handbook at www.lenire.com/product-support/ for headphone specific details).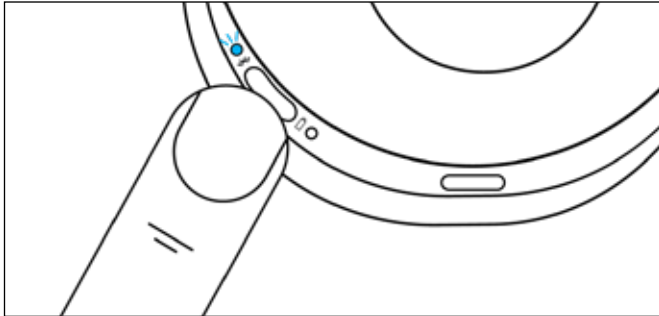


- Switch on the Controller by pressing the Power Button for 2 seconds.
- Place the Controller into Pairing Mode by pressing the Play, Pause and Volume Up [+] Buttons at the same time. All indicator lights will flash blue for 2 seconds.

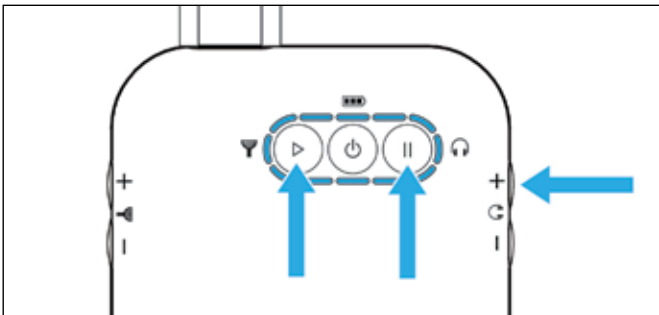- The Controller will search for and attempt to pair with the Headphones. The Headphones Indicator on the Controller will pulse rapidly during this procedure.

- If no discoverable Headphones are found within 20 minutes, the Controller will shut down. Otherwise, the Controller will connect to the Headphones and the Headphones Indicator on the Controller will illuminate green.

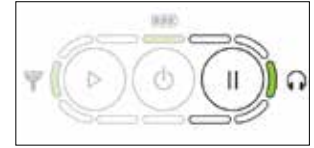Chapter 21 - Servicing

- The device is not intended to be user-serviced. If it should fail for any reason, please return to your prescribing healthcare professional for assistance.
- This device is not intended to be modified in any way.

Chapter 22 - Device Errors

If any of the following error indicators occur that prevents the use of the Lenire device, please contact your prescribing healthcare professional. The errors will be indicated via the lights on the Controller. All but one of the lights will turn solid red. The position of the light that is not lit will indicate the error that has occurred.

After displaying the error indicator for 5 seconds, the Controller will shut down.

Error Indicators

- Controller hardware failure.
- Controller software error.
- Internal storage error.
- Tonguetip hardware failure.
- Tonguetip software error.

Section E

Technical Information

The following section outlines technical information pertaining to the Lenire device. This information is included in order to conform to international safety standards for medical devices. There is also important information included in the following section relating to environmental operating conditions and use in the vicinity of other electrical devices such as mobile phones.

Chapter 23 - Environmental Operating Conditions

Lenire should only be used in environments between 5 and 40 degrees Celsius, relative humidity between 15% and 93%, and atmospheric pressure of between 700 and 1060 hPa. This precludes the use at altitudes above 3000m, unless traveling in an airplane that has the cabin pressurised.

Chapter 24 - Radio Frequency (RF) Specifications

The Lenire Controller contains a Bluetooth wireless radio module that transmits the sound stimulus to the Headphones. If the wireless communication is adversely affected, for example due to obstruction or interference, gaps in sound stimulus may occur. If this should occur during treatment, please try the following:

- Move to a location where likely sources of interference are at least 2 meters (6½ feet) away from the Controller, Tonguetip and Headphones. Likely sources of interference include microwave ovens, Wi-Fi routers, smart watches and mobile phones.
- Ensure there is line-of-sight between the Controller and the Headphones.
- Increase the distance between the Controller and the Headphones. While this may seem counter-intuitive, the Bluetooth system automatically increases the signal power as this distance is increased which may reduce or eliminate gaps in the sound stimulus.

The Bluetooth radio specifications are as follows:

Specification	Description
Frequency Band	2.4 GHz to 2.4835 GHz
Modulation	8 DPSK, PI/4 DQPSK, GFSK
Operating Range (Line-of-Sight)	≥ 10 meters
RF Sensitivity	0.1% BER at -88dBm
Effective Radiated Power (ERP)	<4dBm (2.5 mW) Bluetooth Class 2

Chapter 25 - Guidance and Declaration for Electromagnetic Compatibility

The Lenire Controller, Tonguetip and the Charger have been tested for electromagnetic performance to the requirements of IEC 60601-1-2 which includes testing to the following standards:

- IEC 61000-4-3 (Radiated electromagnetic fields)
 - To 80 MHz to 2.7 GHz at 10 V/m
 - 2.7 GHz to 6 GHz at 3 V/m
 - Spot frequencies per IEC 60601-1-2:2014 Table 9 at up to 28 V/m
- IEC 61000-4-6 (Conducted electromagnetic fields)
 - 150 kHz to 80 MHz at 3V
 - Spot frequencies per IEC 60601-1-2:2014 at 6 V
- IEC 61000-4-2 (Electrostatic Discharge)
 - +/- 8 kV Contact Discharge
 - +/- 15 kV Air Discharge
- IEC 61000-4-8 (50 Hz/60 Hz magnetic fields) at 30 A/m
- IEC 61000-4-4 (Electrical Fast Transients)
- IEC 61000-4-5 (Surge)
- IEC 61000-4-11 (Voltage dips and interruptions)

Disturbances higher than these limits are unlikely to be encountered in the home or professional healthcare environments. Electromagnetic interference may result in temporary disruption of the sound stimulus (manifesting as brief gaps of silence) as described in Chapter 25.

In addition, the Headphones have been tested to the requirements of CISPR 35 (EN 55035), which includes testing to the following standards:

- IEC 61000-4-3 (Radiated electromagnetic fields)
 - 80 MHz to 1 GHz at 3 V/m
 - 1.8 GHz, 2.6 GHz, 3.5 GHz and 5 GHz at 3 V/m
- IEC 61000-4-2 (Electrostatic Discharge)
 - +/- 4 kV Contact Discharge
 - +/- 8 kV Air Discharge

The Lenire device is intended for use in the home healthcare and professional healthcare environments only. The performance of the Lenire in environments with high electromagnetic disturbances, e.g. near magnetic resonance imaging (MRI) or high frequency surgical equipment has not been evaluated and the Lenire should not be used in such environments.

-  Portable radio communications equipment (such as WIFI router, smart watches or mobile phones etc.) may interfere with the performance of the Lenire device (including cables as specified by Neuromod Devices Limited) and should be kept at least 2m away from it when in use. Otherwise, degradation of the performance of this equipment could result.
-  If interference to the performance of the device occurs, such as a change in the display indicators, gaps or drop-outs in the sound, or a noticeable and unusual change in the tongue stimulation, it is possible that a source of electromagnetic interference is in proximity to the device. Please move to a location that significantly reduces the proximity to the electromagnetic interference source.
-  The Lenire device should not be used adjacent to or stacked with other electrical equipment because it could result in improper operation.
-  The Headphones may suffer damage if exposed to Electrostatic Discharges of higher than 8 kV. To minimise the risk of damage, please pick up the Headphones by the headband before use.
-  Use of accessories and cables other than those specified for use for the Lenire device (and as provided by Neuromod Devices Limited) could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
-  The Lenire device should not be used while in proximity to diathermy, electrocautery, magnetic resonance imaging, security or metal detector systems. Otherwise, degradation of the performance of this equipment could result.

Lenire has been tested for electromagnetic emissions and complies with Class B limits of CISPR 11 (Controller and Tonguetip) and CISPR 32 (Headphones) and is therefore unlikely to interfere with the operation of other electrical equipment.

Chapter 26 - Materials in Direct Contact with Human Tissue

The Lenire device does not contain phthalates, latex, human blood derivatives or cells or tissues of human or animal origin.

For a complete list of materials in contact with human tissue, contact Neuromod Devices Limited.

Chapter 27 - Declaration of Conformity Radio Equipment Directive 2014/53/EU

Hereby Neuromod Devices Limited declares that the radio equipment type are Wideband Data Transmission Equipment in the 2.4GHz ISM band (operating in the frequency range 2,4 GHz to 2,4835 GHz per ETSI 300 328 v.2.1.1.) is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: www.neuromoddevices.com.

Chapter 28 - Headphone Specific Technical Information

Use these Headphones only for the intended applications. Austrian Audio cannot accept any liability for damages resulting from improper handling or misuse.

Complies with IEC 62368-1:2018 for mobile audio equipment.

Hereby, Austrian Audio GmbH declares that the radio equipment type Hi-X25BT is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: <https://austrian.audio>

Headphone Manufacturer Contact Details:

Austrian Audio GmbH
Eitnergasse 15, 1230 Vienna, Austria

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Austrian Audio GmbH is under license.

USB Type-C® and USB-C® are registered trademarks of USB Implementers Forum.

Section F

Support Details

Please contact your prescribing healthcare professional for support with your Lenire device.

Neuromod Contact Details:

Neuromod Devices Limited,
Rainsford St,
Dublin,
D08 R2YP, Ireland.

Website: www.lenire.com

Designed & Manufactured by:

Neuromod Devices Limited,
Rainsford St,
Dublin, D08 R2YP,
Ireland.

Note: Any incident that has occurred in relation to the medical device which has resulted in death, temporary or permanent serious deterioration in health or a serious public health threat should be reported to the manufacturer and authority with local jurisdiction.

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