# Lenire® Headphone Handbook

- DA Håndbog til Lenire®-hovedtelefoner
- SV Handbok för Lenire®-hörlurar
- FI Lenire®-kuulokkeiden käsikirja
- NO Lenire® håndbok for hodetelefoner

**LENIRE®**

# Lenire Compatible Headphones

The following document provides details on the use of the compatible Headphones supplied with the Lenire Tinnitus Treatment Device.

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The Lenire device is designed to work with specific Headphones. A list of compatible Headphones that can be provided at the time of purchase by your treating clinician is outlined below.

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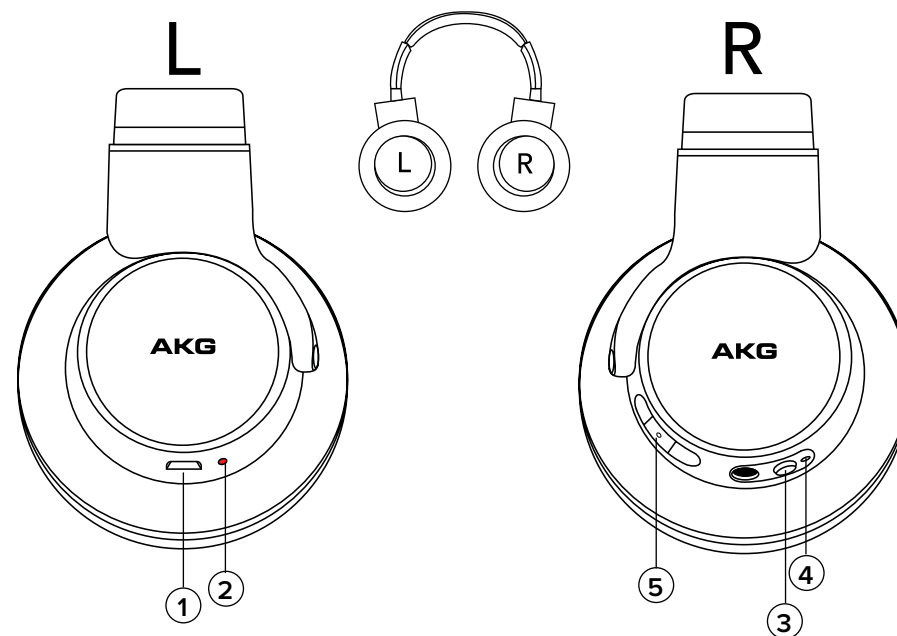
#### 5 Play/Pause & Volume Buttons (not used during treatment)

- DA** - Afspilnings-/pause- og lydstyrkeknapper (bruges ikke under behandling)
- SV** - Spela upp/pausa samt volymknappar (används inte under behandling)
- FI** - Toisto-/pysäytys- ja äänenvoimakkuuspainikkeet (ei käytetä hoidon aikana)
- NO** - Start-/pause- og volumknapper (brukes ikke under behandling)

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**When connected the Headphones Indicator on the Controller will illuminate green.**

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**To disconnect bluetooth connection and enter Headphones into pairing-mode, hold the power button until the Indicator is rapidly flashing blue.**

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**The left and right sides of the Headphones are marked L and R respectively on the cloth inside the foam cushions.**

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**Plug the charging cable into the USB Port on the Headphones.**

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**NO** - Koble ladekabelen til USB-porten på hodetelefonene.

**When charging the Charging Indicator on the Headphones will illuminate red.**

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**NO** - Under ladingen vil ladeindikatoren på hodetelefonene lyse rødt.

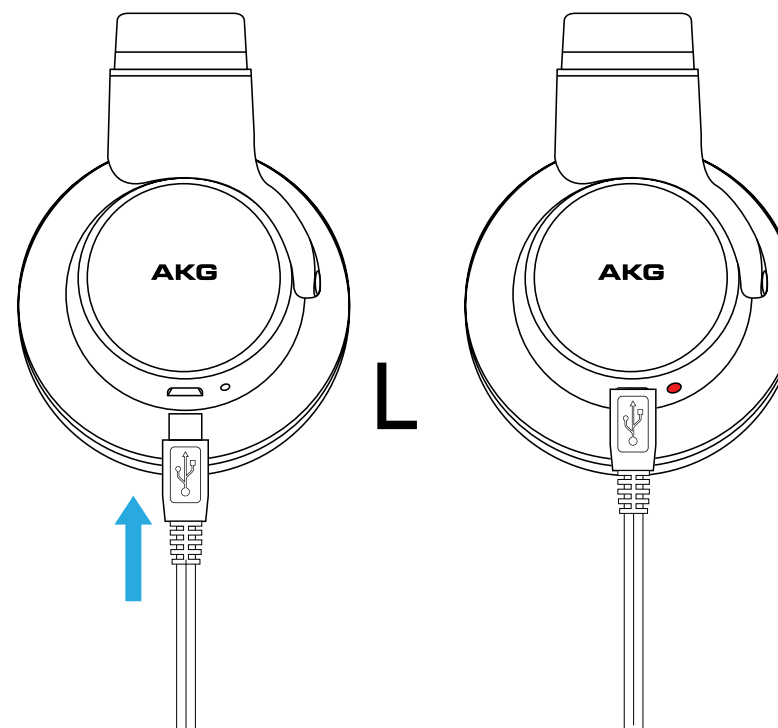
**Once charging is complete, the Charging Indicator on the Headphones will extinguish.**

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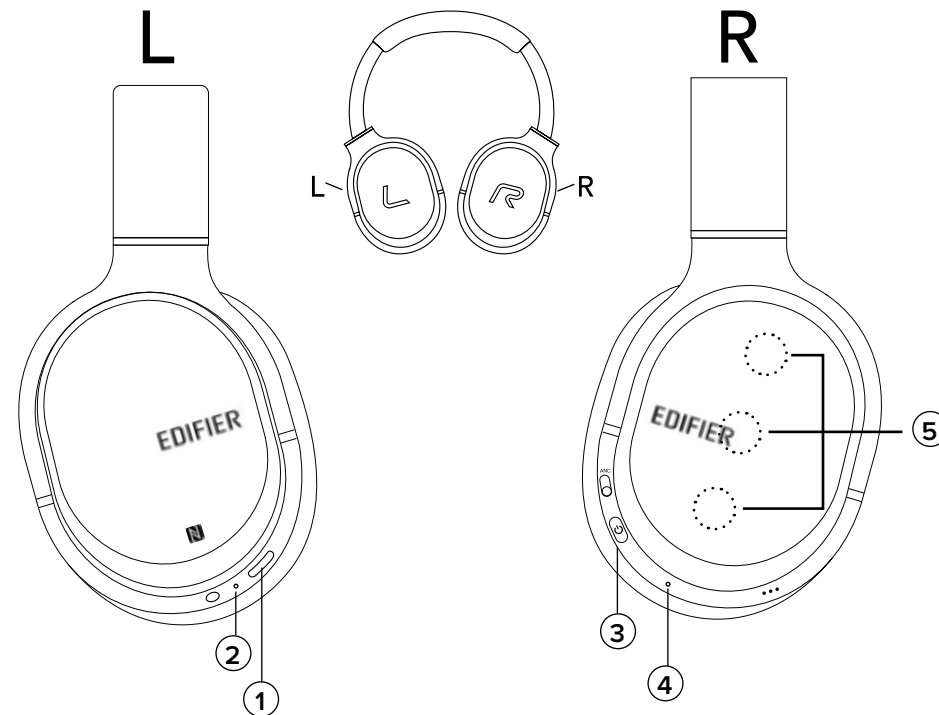
**5 Play/Pause & Volume Buttons (not used during treatment)**

- DA - Afspilnings-/pause- og lydstyrkeknapper (bruges ikke under behandling)
- SV - Spela upp/pausa samt volymknappar (används inte under behandling)
- FI - Toisto-/pysäytys- ja äänenvoimakkuuspainikkeet (ei käytetä hoidon aikana)
- NO - Start-/pause- og volumknapper (brukes ikke under behandling)

**2b : Connecting to the Edifier W860NB / Tilslutning til Edifier W860NB / Anslutning till Edifier W860NB / Edifier W860NB -kuulokkeisiin yhdistäminen / Tilkobling til Edifier W860NB**

**Press the power button and wait for the Bluetooth Indicator to flash blue.**

- DA** - Tryk på tænd/sluk-knappen, holde den nede i 3 sekunder, til Bluetooth-indikatoren blinker blåt.  
**SV** - Tryck på strömbrytaren i 3 sekunder och vänta tills Bluetooth-indikatorn blinkar blått.  
**FI** - Paina virtapainiketta kolme sekuntia ja odota, että Bluetooth-merkkivalo alkaa vilkkua sinisenä.  
**NO** - Hold inne strømknappen i 3 sekunder og vent til Bluetooth-indikatoren blinker blått.

**Hold the Controller close to the Headphones.**

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**When connected the Headphones Indicator on the controller will illuminate green.**

- DA** - Efter tilslutning lyser hovedtelefonindikatoren på controlleren grønt.  
**SV** - När hörlurarna är anslutna lyser hörlursindikatorn på kontrollenheten grönt.  
**FI** - Kun yhteys on muodostettu, ohjaimessa oleva kuulokkeiden merkkivalo alkaa palaa vihreänä.  
**NO** - Etter tilkobling vil hodetelefonindikatoren på kontrolleren lyse grønt.

**2c : Entering Pairing/Re-pairing mode / Åbning af parrings-/genparringstilstand / Gå in i läget för parning/omparning / Laiteparitilaan siirtyminen / Gå inn i modus for parkobling/gjenta parkobling**

**To disconnect bluetooth connection and enter headphones into pairing-mode, hold the power button until the indicator is rapidly flashing blue and red.**

- DA** - For at afbryde bluetooth-forbindelsen og sætte hovedtelefonerne i parringstilstand skal du holde tænd/ sluk-knappen nede, indtil indikatoren blinker blåt og rødt hurtigt.  
**SV** - För att koppla bort Bluetooth-anslutningen och sätta hörlurarna i parningsläge håller du in strömbrytaren tills indikatorn blinkar snabbt blått och rött.  
**FI** - Jos haluat katkaista Bluetooth-yhteyden ja palauttaa kuulokkeet laiteparitilaan, pidä virtapainiketta pohjassa, kunnes merkkivalo vilkkuu nopeasti sinisenä ja punaisena.  
**NO** - For å koble fra Bluetooth-tilkoblingen og sette hodetelefonene i parkoblingsmodus holder du inne strømknappen frem til indikatoren blinker raskt i blått og rødt.

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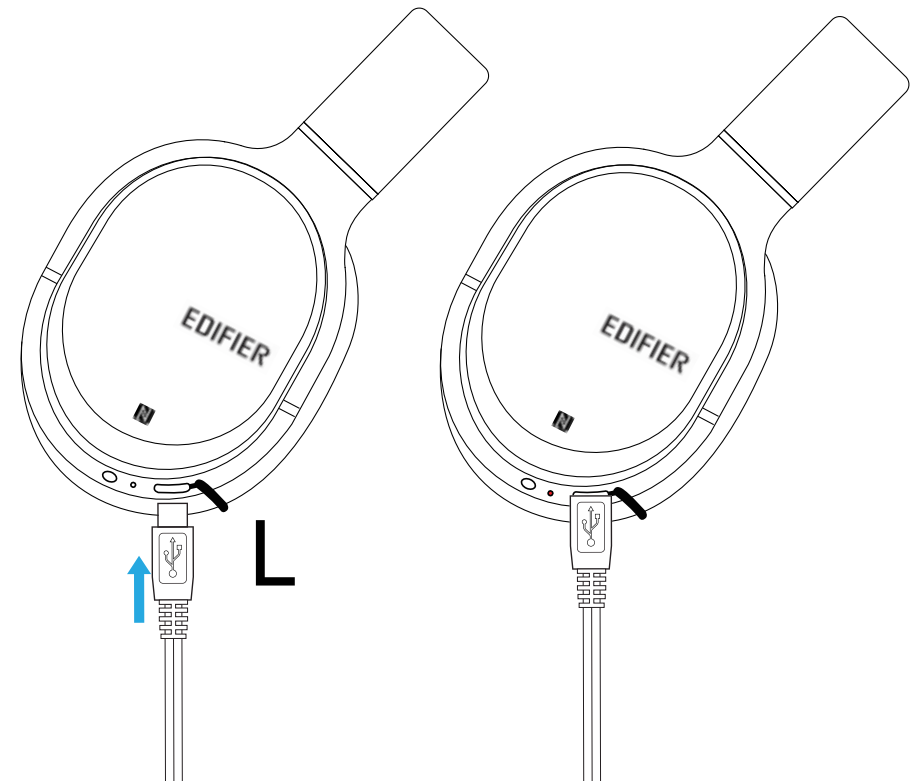
# Customer Support Contact Details

## General Contact Details

Neuromod Devices Limited,  
Rainsford St,  
Dublin,  
D08 R2YP, Ireland.

**E-mail:** [customersupport@neuromoddevices.com](mailto:customersupport@neuromoddevices.com)

**Web:** [www.neuromoddevices.com](http://www.neuromoddevices.com)

## Technical Support Contact Details

**E-mail:** [techsupport@neuromoddevices.com](mailto:techsupport@neuromoddevices.com)

**Web:** [www.neuromoddevices.com](http://www.neuromoddevices.com)