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English			
Introduction			
Your healthcare professional has prescribed the Lenire® Device for your use to alleviate the symptoms of chronic, subjective tinnitus. Please read the Lenire User Manual carefully before use. These instructions will show you how to use and care for your Tonguetip® TP-101. The Tonguetip TP-101 should only be used in conjunction with the Lenire Device.	IP67 of the Tonguetip contain nickel. You may experience a mild numbness sensation or a discomfort on the tongue during treatment. If this occurs, reduce the tongue stimulation. If the symptoms persist, contact your prescribing healthcare professional.	How to use your Tonguetip	Tonguetip service life
Intended use	Warnings <ul style="list-style-type: none"> Do not use the Tonguetip when going to sleep or under the influence of alcohol or other substances that cause drowsiness. Keep the Tonguetip out of the reach of children, to avoid possible choking and strangulation hazards. To avoid the risk of cross-contamination, the Tonguetip should not be shared with any other person. If you have lesions, sores, or inflammation of the oral cavity, discontinue use of the device until symptoms have resolved. 	How the Tonguetip works The Tonguetip is a medical device accessory that, when connected to a Lenire Controller, delivers gentle, electrical stimulation to the tongue through electrodes which are placed on the tip of your tongue. Your clinician will set the stimulation level to your individual setting.	<ul style="list-style-type: none"> The Tonguetip has an expected service life of three years. It is recommended not to use the Tonguetip after this as the electrodes may not function as effectively. The Tonguetip will expire after 180 cumulative hours of actual use. If an expired Tonguetip is connected to the Controller, the Tonguetip indicator will flash red.
Contraindications	Package labelling <ul style="list-style-type: none"> MD Medical Device. ⚠️ Do not use if the packaging is damaged or previously opened. i Consult the User Manual. ⚠️ Warnings and precautions associated with this medical device. ⚠️ Manufacturer's details. LOT The device Lot Number. REF The device Model Number. 	Technical description Please read the Lenire user manual for the device technical description.	Cleaning <ul style="list-style-type: none"> Before cleaning the Tonguetip ensure that it is disconnected from the Controller. Do not expose the plug end of the Tonguetip to liquids.